


How to fit and remove a disposable face mask

Fitting mask:

Step 1: Clean your hands with soap and water or hand sanitiser.



Step 2: Hold your mask by the ear loops. Check the pleats to ensure that the folds on the outside of the mask point downwards.

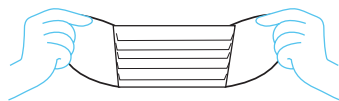

Step 3: Place securely behind your ears with the loops.



Step 4: Once the mask is in place, use your index finger and thumb to pinch the bendable top edge of the mask around the bridge of your nose.




Step 5: Pull the bottom of the mask over your chin.




Removing and disposing of mask:

Step 6: With clean hands, remove loops from ears.

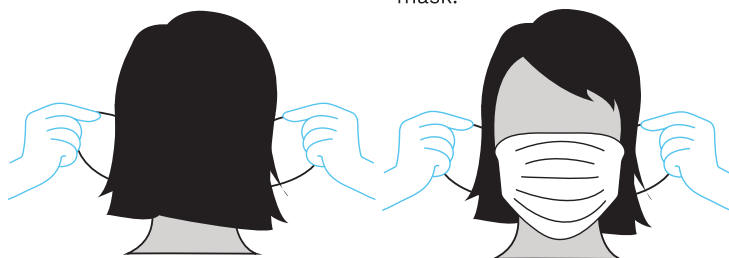

Step 7: Do not touch the front of the mask, eyes, nose and mouth when removing your mask.

Step 8: Discard in appropriate waste receptacle.

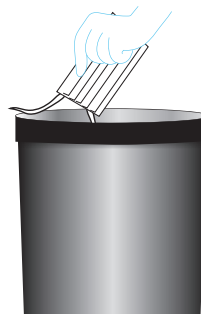

Step 9: Clean your hands with soap and water or hand sanitiser.



Reynard is reassurance.



Reynard
Health Supplies

Australia

44 Hampden Road
Artarmon, NSW, 2064
enquiries@reynardhealth.com.au
1300 667 199

New Zealand

PO Box 8470
Havelock North
Sales@reynardhealth.com
0508 494 737

UK

13 White Horse Business Park,
Stanford in the vale SN7 8NY
uk@reynardhealth.com
+44 (0)845 263 7335

International

PO Box 8470
Havelock North
Sales@reynardhealth.com
+64 (0)6 650 0709