

# Reynard Premier detergent & disinfectant wipes

A single step solution to cleaning and disinfecting surfaces and equipment, proven to kill at least 99.999% of pathogens. With a superior cloth and unsurpassed equipment compatibility, germs don't stand a chance.

## Specifications

- Effective for use against COVID and SARS
- Unparalleled equipment compatibility
- Kills 99.999% of pathogens & bacteria

## Points to remember

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom

Step 1: Remove plastic tamper proof seal strip from under the tub lid and discard.



Step 2: Remove lid by easing it off area by area in a circular motion.



Step 3: Open the sealed bag containing the wipes (do not remove the cloths from the bag).



Step 4: Take hold of the wipe in the centre of the roll and pull through dispensing hole in the tub lid.



Step 5: Re-attach the lid to the tub.




Step 6: Pull the required number of wipes through the dispensing hole.



Step 7: Pull sharply in a downward motion to detach the wipe(s) from the roll, ensuring that the top of the next wipe is presented outside the dispensing hole for next use.



Step 8: Always reseal lid to keep the product moist.

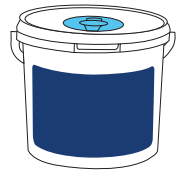

Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

SKU	WIPES PER TUB	TUBS PER CARTON
RHS205	280	2



Reynard is reassurance.



**Reynard**  
Health Supplies

### Australia

44 Hampden Road  
Artarmon, NSW, 2064  
enquiries@reynardhealth.com.au  
1300 667 199

### New Zealand

PO Box 8470  
Havelock North  
Sales@reynardhealth.com  
0508 494 737

### UK

13 White Horse Business Park,  
Stanford in the vale SN7 8NY  
uk@reynardhealth.com  
+44 (0)845 263 7335

### International

PO Box 8470  
Havelock North  
Sales@reynardhealth.com  
+64 (0)6 650 0709