

Reynard Premier Detergent & disinfectant wipes.

A single step solution to cleaning and disinfecting surfaces and equipment. Proven to kill at least 99.999% of pathogens and viruses including SARS and COVID. With unsurpassed equipment compatibility and a superior cloth that is also available in a 100% biodegradable version.

Points to remember:

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom
- Always reseal the pack after use

Step 1: Remove plastic tamper-proof seal strip from under the tub lid and discard.



Step 2: Remove lid by easing off area by area in a circular motion.



Step 3: Open the sealed bag containing the wipes (do not remove the cloths from the bag).



Step 4: Take hold of the wipe in the centre of the roll and pull through the dispensing hole in the tub lid.



Step 5: Reattach the lid to the tub.



Step 6: Pull the required number of wipes through the dispensing hole.



Step 7: Pull sharply in a downward motion to detach the wipe(s) from the roll, ensuring that the top of the next wipe is presented outside the dispensing hole for next use.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

Product code	RHS205
Wipes per canister	280
Canisters per carton	2



Reynard is reassurance.

