

Reynard

Neutral detergent wipes

An alcohol- and fragrance-free solution to effectively cleanse hard surfaces and equipment with no risk of tainting. Widely used in the health sector for cleaning mattress covers, operating theatres and other areas where harsh chemicals need to be avoided.

Specifications

- Unparalleled equipment compatibility
- Cleans surfaces without drying, spotting or leaving sticky residue

Points to remember

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom

Step 1: Remove plastic tamper proof seal strip from under the tub lid and discard.



Step 2: Remove lid by easing it off area by area in a circular motion.



Step 3: Open the sealed bag containing the wipes (do not remove the cloths from the bag).



Step 4: Take hold of the wipe in the centre of the roll and pull through dispensing hole in the tub lid.



Step 5: Re-attach the lid to the tub.



Step 6: Pull the required number of wipes through the dispensing hole.



Step 7: Pull sharply in a downward motion to detach the wipe(s) from the roll, ensuring that the top of the next wipe is presented outside the dispensing hole for next use.



Step 8: Always reseal lid to keep the product moist.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

SKU	WIPES PER TUB	TUBS PER CARTON
RHS206	280	2



Reynard is reassurance.



Reynard
Health Supplies

New Zealand

PO Box 8470
Havelock North
Sales@reynardhealth.com
0508 494 737

Australia

44 Hampden Road
Artarmon, NSW, 2064
enquiries@reynardhealth.com.au
1300 667 199

UK

White Horse Business Park,
Standford in the Vale SN7 8NY
uk@reynardhealth.com
+44 (0)845 263 7335

International

PO Box 8470
Havelock North
Sales@reynardhealth.com
+64 (0)6 650 0709