


# Reynard Premier detergent & disinfectant wipes.

A single step solution to cleaning and disinfecting surfaces and equipment. Proven to kill at least 99.999% of pathogens and viruses including SARS and COVID. With unsurpassed equipment compatibility and a superior cloth that is also available in a 100% biodegradable version.


## Points to remember:

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom
- Always reseal the pack after use


Step 1: Peel back the sticky pack cover, take hold of a wipe and pull it out of the pack.

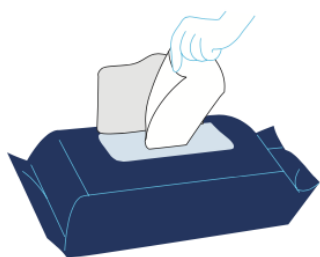

Step 2: Open wipes to desired size and reseal cover to retain moisture.

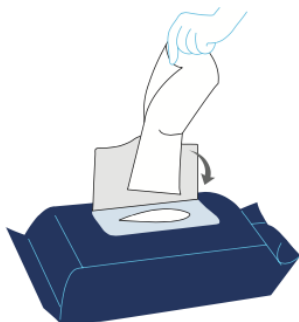

Step 3: Wipe surface in an 'S' shaped motion from top to bottom.

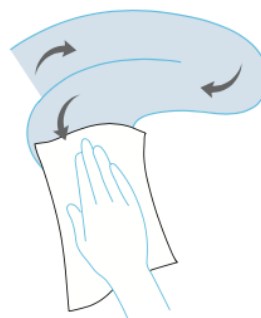

Step 4: Dispose of wipe. Do not flush or put into macerator.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.



Product code	RHS210	RHS216	RHS218
Wipes per pack	50	50	100
Packs per carton	12	12	6
Biodegradable	Yes	No	No

Reynard is reassurance.

