

Reynard Premier detergent & disinfectant wipes

A single step solution to cleaning and disinfecting surfaces and equipment, proven to kill at least 99.999% of pathogens. With a superior cloth and unsurpassed equipment compatibility, germs don't stand a chance.


Specifications

- Effective for use against COVID and SARS
- Unparalleled equipment compatibility
- Superior cloth: up to 47% larger than competitors


Points to remember

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom


Step 1: Peel back the sticky pack cover and take hold of a wipe and pull it out of the pack

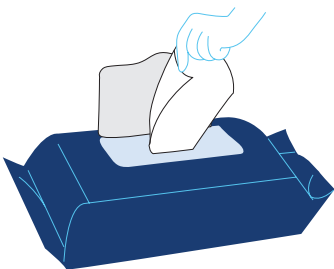


Step 2: Open wipe to desired size and re-seal cover to retain moisture.

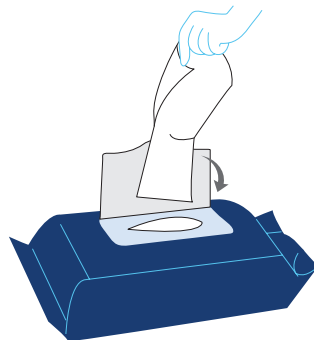

Step 3: Wipe surface in an 'S' shaped motion from top to bottom.

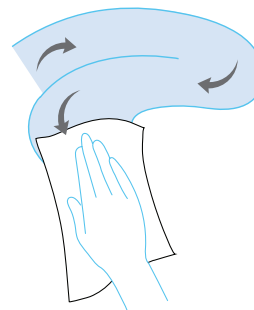

Step 4: Dispose of wipe. Do not flush or put into a macerator.

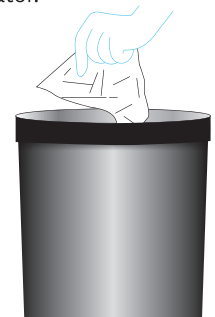

Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.



SKU	WIPES PER PACK	PACKS PER CARTON
RHS216	50	100
RHS218	100	6

Reynard is reassurance.



Australia

44 Hampden Road
Artarmon, NSW, 2064
enquiries@reynardhealth.com.au
1300 667 199

New Zealand

PO Box 8470
Havelock North
Sales@reynardhealth.com
0508 494 737

UK

13 White Horse Business Park,
Stanford in the vale SN7 8NY
uk@reynardhealth.com
+44 (0)845 263 7335

International

PO Box 8470
Havelock North
Sales@reynardhealth.com
+64 (0)6 650 0709