


Reynard Clinical Skin Prep

Prep Pads: Instructions for use




Step 1: Check allergy adverse reaction status with your patient

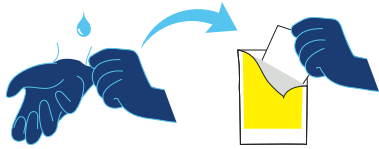


Step 2: Peel open sachet and remove prep pad using an Aseptic Non-Touch Technique (ANTT)



Step 3: If applicable, check that the antiseptic is compatible with the catheter material

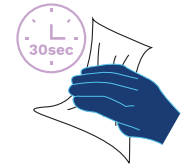

Step 4: Wipe the skin in a circular motion to cleanse prior to injection or cannulation for thirty (30) seconds

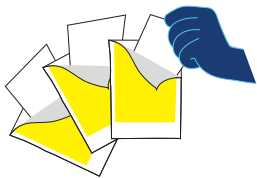

Step 5: Use the required number of prep pads to ensure adequate coverage dependent on the clinical procedure



Step 6: Allow the skin area to air dry completely. Under normal conditions this should take approximately thirty seconds



Step 7: Prep pads are single use only. Dispose of prep pads accordance with your facility's Infection Control Guidelines and waste management protocols, do not flush.



Further information

a. Refer to your facilities "IV Therapy and Related Procedures Manual" for management of a peripheral IV cannula, venipuncture and accessing luer plugs for further information

Reynard is reassurance.



Reynard
Health Supplies

New Zealand

PO Box 8470
Havelock North
Sales@reynardhealth.com
0508 494 737

Australia

44 Hampden Road
Artarmon, NSW, 2064
enquiries@reynardhealth.com.au
1300 667 199

UK

White Horse Business Park,
Standford in the Vale SN7 8NY
uk@reynardhealth.com
+44 (0)845 263 7335

International

PO Box 8470
Havelock North
Sales@reynardhealth.com
+64 (0)6 650 0709