

Reynard Premier Detergent & disinfectant wipes.

A single step solution to cleaning and disinfecting surfaces and equipment. Proven to kill at least 99.999% of pathogens and viruses including SARS and COVID. With unsurpassed equipment compatibility and a superior cloth that is also available in a 100% biodegradable version.

Points to remember:

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom
- Always reseal the pack after use

Step 1: Turn the lid in a clockwise motion to remove.



Step 2: Cut two crossing slits into the foil seal, approximately 3cm in length each.



Step 3: Take hold of the wipe in the centre of the roll and pull through the cross.



Step 4: Thread the cloth through the dispensing hole in the lid.



Step 5: Reattach the lid to the canister.



Step 6: Pull the required number of wipes through the dispensing hole.



Step 7: Pull sharply in a downward motion to detach the wipe(s) from the roll, ensuring the top of the next cloth is presented



Step 8: Always reseal the lid to keep product moist.



outside the dispensing hole for the next use.

Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

Product code	RHS219
Wipes per canister	200
Canisters per carton	12



Reynard is reassurance.

