

Reynard IPA wipes

Hospital grade disinfectant.

Fragrance-free and non-tainting, these 70% isopropyl alcohol (IPA) wipes are ideal for fast and effective disinfection of most hard surfaces and equipment. Proven to be effective against most illness causing germs including MRSE and VRE.

Points to remember:

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom
- Always reseal the pack after use

Step 1: Turn the lid in a clockwise motion to remove.




Step 2: Cut two crossing slits into the foil seal, approximately 3cm in length each.



Step 3: Take hold of the wipe in the centre of the roll and pull through the cross.



Step 4: Thread the cloth through the dispensing hole in the lid.

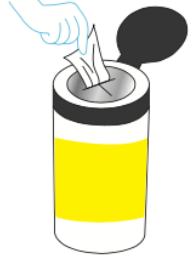

Step 5: Reattach the lid to the canister.



Step 6: Pull the required number of wipes through the dispensing hole.



Step 7: Pull sharply in a downward motion to detach the wipe(s) from the roll, ensuring the top of the next cloth is presented outside the dispensing hole for the next use.



Step 8: Always reseal the lid to keep product moist.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

Product Code	RHS402	RHS409
Wipes per canister	160	75
Canisters per carton	12	12
Wipe size	22 x 20cm	42 x 14cm



Reynard is reassurance.

