

Reynard

Neutral detergent wipes

An alcohol- and fragrance-free solution to effectively cleanse hard surfaces and equipment with no risk of tainting. Widely used in the health sector for cleaning mattress covers, operating theatres and other areas where harsh chemicals need to be avoided.


Specifications

- Unparalleled equipment compatibility
- Cleans surfaces without drying, spotting or leaving sticky residue


Points to remember

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom


Step 1: Peel back the sticky pack cover and take hold of a wipe and pull it out of the pack

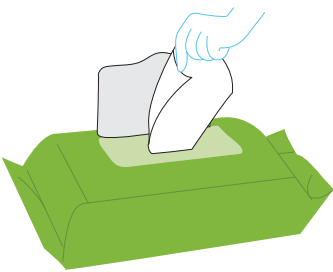


Step 2: Open wipe to desired size and re-seal cover to retain moisture.

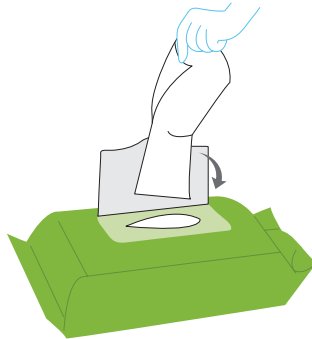

Step 3: Wipe surface in an 'S' shaped motion from top to bottom.

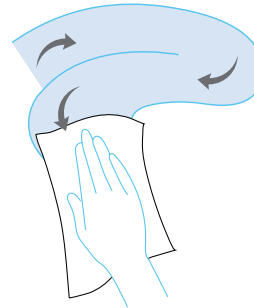

Step 4: Dispose of wipe. Do not flush or put into a macerator.

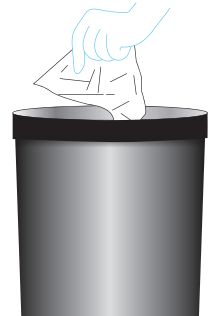

Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.



SKU	WIPES PER PACK	PACKS PER CARTON
RHS201	50	12

Reynard is reassurance.



New Zealand
PO Box 8470
Havelock North
Sales@reynardhealth.com
0508 494 737

Australia
44 Hampden Road
Artarmon, NSW, 2064
enquiries@reynardhealth.com.au
1300 667 199

UK
White Horse Business Park,
Standford in the Vale SN7 8NY
uk@reynardhealth.com
+44 (0)845 263 7335

International
PO Box 8470
Havelock North
Sales@reynardhealth.com
+64 (0)6 650 0709