

Reynard Neutral detergent wipes.

Unparalleled equipment compatibility.

An alcohol and fragrance-free solution to effectively cleanse hard surfaces and equipment with no risk of tainting. Widely used in the health sector for cleaning medical equipment, mattress covers, operating theatres and other areas where harsh chemicals need to be avoided.

Points to remember:

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom
- Always reseal the pack after use


Step 1: Peel back the sticky pack cover, take hold of a wipe and pull it out of the pack.



Step 2: Open wipes to desired size and reseal cover to retain moisture.



Step 3: Wipe surface in an 'S' shaped motion from top to bottom.

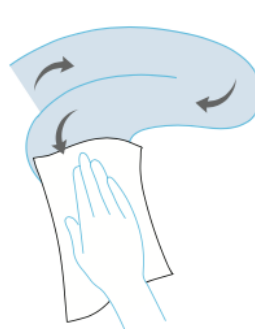

Step 4: Dispose of wipe. Do not flush or put into macerator.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

Product code	RHS201
Wipes per pack	50
Packs per carton	12



Reynard is reassurance.

