

Reynard IPA Wipes

Hospital grade disinfectant

Fragrance-free and non-tainting, these 70% isopropyl alcohol wipes are ideal for fast and effective disinfection of most hard surfaces and equipment.

Specifications

- Proven to be effective against 99% of common germs including MRSA and VRE
- The original disinfection wipes

Points to remember

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom

Step 1: Turn lid in a clockwise motion to remove.




Step 2: Cut two crossing slits into the foil seal, approximately 3cm in length each.



Step 3: Take hold of the wipe in the centre of the roll and pull through the cross.



Step 4: Thread the cloth through the dispensing hole in the lid.

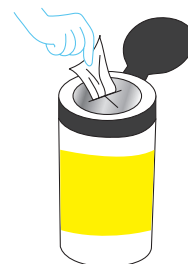

Step 5: Re-attach the lid to the canister.



Step 6: Pull the required number of wipes through the dispensing hole.



Step 7: Pull sharply in a downward motion to detach the wipe(s) from the roll, ensuring the top of next cloth is presented outside the dispensing hole for next use.



Step 8: always re-seal lid to keep product moist.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

SKU	WIPES PER TUB	TUBS PER CARTON
RHS402	160	12
RHS480	150	12



Reynard is reassurance.



Reynard
Health Supplies

New Zealand

PO Box 8470
Havelock North
Sales@reynardhealth.com
0508 494 737

Australia

44 Hampden Road
Artarmon, NSW, 2064
enquiries@reynardhealth.com.au
1300 667 199

UK

White Horse Business Park,
Standford in the Vale SN7 8NY
uk@reynardhealth.com
+44 (0)845 263 7335

International

PO Box 8470
Havelock North
Sales@reynardhealth.com
+64 (0)6 650 0709