

Reynard Warmer Cabinet

A convenient storage and preparation unit designed to keep products ready for immediate use at the correct temperature. The warmer allows users to safely and efficiently warm wipes for the benefit and comfort of the patient.



Status Indicator:

The warmer has a heating status display system designed to indicate which wipes are ready to use:

Heating – Red light

Packs have not yet reached the required temperature.

Heated – Green light

Packs have reached the required temperature and are ready to use.

Use First – Green flashing light

Pack has been in the warmer for 60 hours and should be used first.

- 1 Place the packet in the cabinet slot, then close the door
- 2 The status lights are activated when a wipe package is placed in a slot within the warmer
- 3 The warmer will heat up to the adjusted temperature and maintain this temperature (unless it is changed or the warmer is switched off)
- 4 The status indicator will alert the user when a package has reached the target temperature
- 5 The warmer is also capable of indicating which packages have been in the warmer for an extended period of time and therefore should be used first. This aids in package stock rotation so product remains fresh

PRODUCT	SKU	CH2 SKU
Bath in Bed Wipes	RHS102	1855166
Shampoo Cap	RHS104	1855182
Chlorhexidine Wash Cloths	RHS107	2071339

Please note: Products will take between 90-140 minutes to heat, depending on the temperature set.

Always check the temperature of the cloth before use.

Reynard is reassurance.



Reynard
Health Supplies

New Zealand

PO Box 8470
Havelock North
Sales@reynardhealth.com
0508 494 737

Australia

44 Hampden Road
Artarmon, NSW, 2064
enquiries@reynardhealth.com.au
1300 667 199

UK

White Horse Business Park,
Standford in the Vale SN7 8NY
uk@reynardhealth.com
+44 (0)845 263 7335

International

PO Box 8470
Havelock North
Sales@reynardhealth.com
+64 (0)6 650 0709