

Reynard™ Warmer cabinet.

How to use.

Step 1: Place the packet in a cabinet slot, then close the door.

Step 2: The status light will activate when a wipe package is placed in a slot within the warmer.

Step 3: The status indicator will alert the user when a package has reached the target temperature.

Status Indicator

The warmer has a heating status display system designed to indicate which wipes are ready to use:

Heating – Red light

Packs have not yet reached the required temperature.

Heated – Green light

Packs have reached the required temperature and are ready to use.

Use First – Green flashing light

Pack has been in the warmer for 60 hours and should be used first.



The flashing light will indicate which products have been in the warmer for an extended period, and therefore should be used first. This aids in stock rotation so the product remains fresh.

Reynard™ warmers are intended for use with the Reynard™ following products:

| Product | SKU | CH2 SKU |
|---|--------|---------|
|  Bath in bed wipes | RHS102 | 1855166 |
|  Shampoo cap | RHS104 | 1855182 |
|  Chlorhexidine wash cloths | RHS107 | 2071339 |

Please note: products will take between 90-140 minutes to heat, depending on the temperature set. Always check the temperature of the product before use.

Reynard™ is reassurance.

